


HC2001B (REMOTE CONTROL)

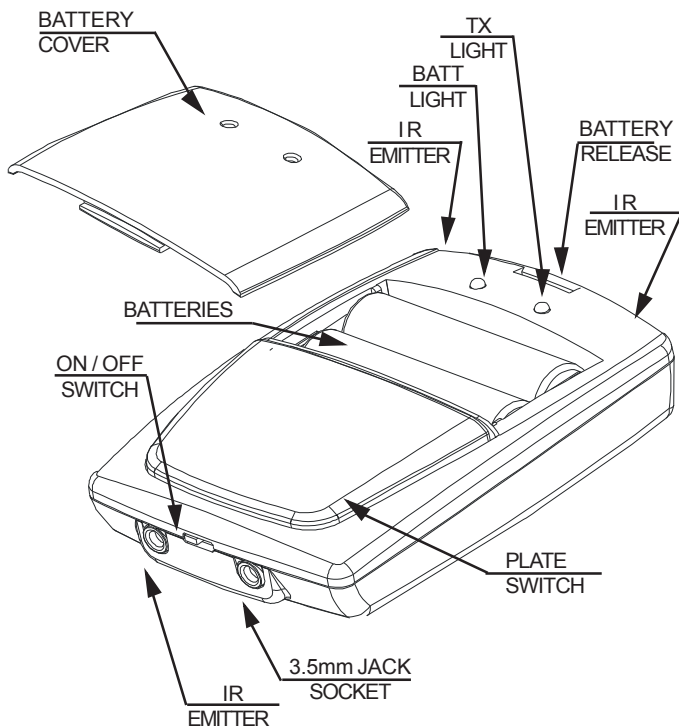

The HC2001B remote control unit is designed to work with a Possum 3.5mm mono input switch. The HC2001B acts as a wireless link when the input is pressed. Infra-red is transmitted and a FREEWAY controller is activated. The unit incorporates a plate switch which may be used by the carer/user.

INDICATOR LIGHTS

The small green light labelled TX illuminates when an infra-red transmission is activated. The small yellow light labelled BATT illuminates when batteries are low, replace batteries as soon as possible.

THE POWER SWITCH

The switch (indicated as ON/OFF) on the back of the HC2001B turns the unit on or off. The unit operates from its internal battery and this switch will save power if it is switched off.

SECURING UNIT

The HC2001B is supplied with velcro attached to the unit for your securing if required. Please note that once attached this velcro can be difficult to remove.

PRODUCT CARE

1. Switch off and disconnect any external cables that are plugged into the HC2001B unit before attempting to clean the product.
2. The Possum HC2001B should be cleaned only with a soft, lint-free cloth slightly moistened with clean water.
3. Do not use abrasive or solvent-based cleaning products.
4. Do not allow moisture to enter the HC2001B unit. Ensure that the product is dry before reconnecting any external leads.
5. The Possum HC2001B is intended for indoor use only and is not drip proof, splash proof or waterproof.
6. The Possum HC2001B is not anaesthetic proof and cannot be sterilised.

IF YOU HAVE A PROBLEM

IF THE UNIT STOPS RESPONDING

If the unit fails to respond then check if the IR Detectors on the HC2001B are not obstructed from the controller.

If there is still no response then change the batteries within the unit.

To change the batteries, push in pip on front of case and take off the grey top to reveal the battery compartment with two AA 1.5V cells (see diagram).

These *MUST* be replaced with AA 1.5V minimum 2800mAh cells or Possum part 355-005.

(DO NOT USE RECHARGEABLE BATTERIES)

Check the polarity of the batteries as you remove them and place the new batteries in the same way.

The unit should now function correctly.

PLEASE NOTE:

If you are unable to solve the problem, please contact Possum Controls Ltd. at the address overleaf for assistance.

HC2001B

POSSUM CONTROLS LIMITED
8, FARMBROUGH CLOSE
STOCKLAKE PARK INDUSTRIAL EST
AYLESBURY
BUCKS.
HP20 1DQ

TELEPHONE 01296 461000
FAX 01296 461001

email sales@possum.co.uk
WEBSITE <http://www.possum.co.uk>

Notice

Possum Controls Limited reserves the right to make improvements to this guide or the products it describes at any time and without further notice. This guide is copyright in the U.K. and all rights are reserved. No part of this guide may be reproduced by any means or in any form, without prior consent in writing from Possum Controls Limited.

HC2001B User Guide - part number: 034-484 Iss:1

© Possum Controls Limited, June 2005



possum
helping independence emerge

SCOPE OF USE

Possum Controls Limited products are not authorised for use as critical components in life support devices or systems which sustain life and whose failure to perform, when properly used in accordance with the instructions for use provided, can reasonably be expected to result in significant injury or death. A critical component is any component of a life support device or system whose failure to perform can reasonably be expected to cause the failure of the life support device or system or to affect its safety or effectiveness.

APPROVALS

The Possum HC2001B is CE marked and made to

 EN60601-1-1 (electrical safety)
EN60601-1-2 (electro-magnetic compatibility)

This equipment does not generate, use or radiate radio frequency energy. It should not cause interference with radio and television receivers or other sensitive equipment. This equipment may be susceptible to radio frequency energy or mains conducted interference generated by other nearby equipment. Interference is unlikely to be a problem in a typical domestic environment, but if you have any queries please contact Possum Controls Limited for advice.

SERVICING

The only parts that are user-serviceable within the Possum HC2001B are the batteries. Refer all faults to Possum Controls Limited or their authorised service agents. For battery replacement see relevant section.

SPECIFICATIONS INTERNAL BATTERIES

The HC2001B contains 2 AA 1.5V minimum 2800mAh battery cells.
These can be changed by a responsible person.
Do not destroy old cells.
Dispose of old battery cells at an appropriate facility.
Batteries will be changed by an Engineer at the annual service.
Note: batteries should NOT be returned to Possum Controls Limited.

Please Note: *These batteries are NOT rechargeable. Do NOT attempt to recharge these batteries.*

OPERATING/STORAGE ENVIRONMENT

Storage Temperature Range: 0°C to 40°C
(32°F to 104°F)
Storage Humidity Range: 30% to 75%
(non-condensing)
Storage Altitude Range: 0m to 2,000m

WEIGHT AND PHYSICAL DIMENSIONS

Width: 70mm
Length: 100mm
Depth: 30mm
Weight: 135 grammes